RadiForce® MX215

Color LCD Monitor

Instructions for Use: Setup Manual

Important: Please read PRECAUTIONS, this Setup Manual and the User's Manual (on the CD-ROM) carefully to familiarize yourself with safe and effective usage. Please retain this manual for future reference.

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Compatible Resolutions

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The monitor supports the following resolutions.

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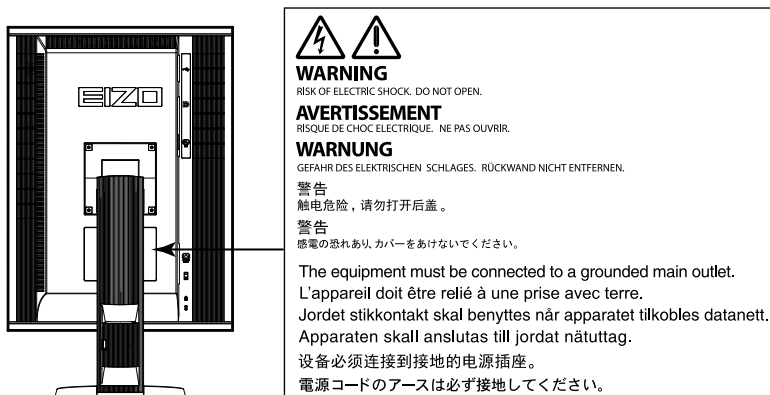
RadiForce® MX215

Color LCD Monitor

Location of the Caution Statements

Symbols on the unit

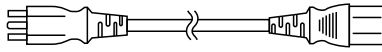
Symbol	This symbol indicates	
	Main Power Switch:	Press to turn the monitor's main power off.
	Main Power Switch:	Press to turn the monitor's main power on.
	Power button:	Press to turn the monitor's power on or off.
	Alternating current	
	Alerting electrical hazard	
	CAUTION:	Refer to SAFETY SYMBOLS section in PRECAUTIONS.
	WEEE marking:	Product must be disposed of separately; materials may be recycled.
	CE marking:	EU conformity mark in accordance with the provisions of Council Directive and/or Regulation (EU).
	Manufacturer	
	Date of manufacture	

Package Contents

Check that all the following items are included in the packaging box. If any items are missing or damaged, contact your local dealer.

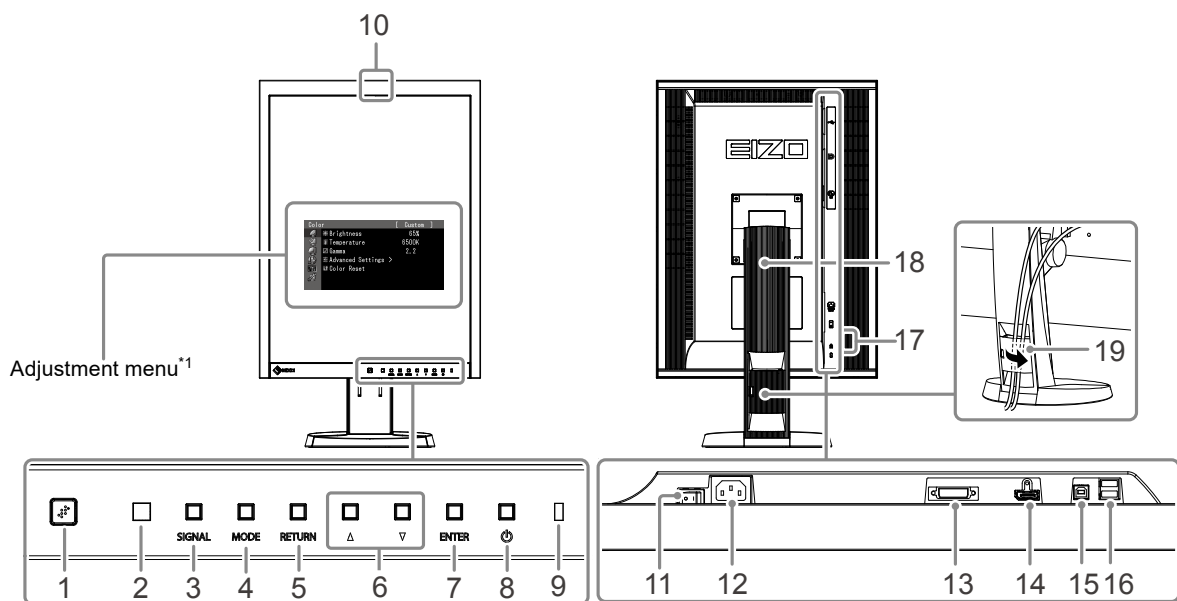
Note

- Please keep the packaging box and materials for future movement or transport of the monitor.
 - Monitor
 - Power cord
 - USB cable: UU300
 - EIZO LCD Utility Disk (CD-ROM)
 - Setup Manual (this manual)
 - PRECAUTIONS

- Digital signal cable: DisplayPort - DisplayPort (PP300)
- Digital signal cable: DVI-D - DVI-D (DD300)

Controls and Functions

1. Presence Sensor	Detects the movement of a person in front of the monitor.
2. Ambient light sensor	Used for the illuminance change monitoring function of RadiCS/RadiCS LE. For details, refer to RadiCS/RadiCS LE User's Manual (on the CD-ROM).
3. SIGNAL button	Switches input signals for display.
4. MODE button	Switches the CAL Switch mode.
5. RETURN button	Cancels the setting/adjustment and exits the Adjustment menu.
6. ▲▼ button	Provides the menu selection as well as the adjustment and setting of a function.
7. ENTER button	Displays the Adjustment menu, determines an item on the menu screen, and saves values adjusted.
8. ⏻ button	Turns the power on or off.
9. Power indicator	Indicates the monitor's operation status. Green: Operating, Orange: Power saving mode, Off: Main power/power off
10. Integrated Front Sensor (slide type)	Used for calibration and Grayscale Check.
11. Main power switch	Turns the main power on or off.
12. Power connector	Connects the power cord.
13. Input signal connector	DVI-I connector
14. Input signal connector	DisplayPort connector
15. USB upstream port	Connects the USB cable to use the software that needs USB connection, or to use USB Hub function.
16. USB downstream port	Connects a peripheral USB device.
17. Security lock slot	Complies with Kensington's MicroSaver security system.
18. Stand	Used to adjust the height and angle of the monitor screen.
19. Cable holder	Covers the monitor cables.

*1 For instructions for use, refer to the User's Manual (on the CD-ROM).

Connecting Cables

Attention

- Check that the monitor and the PC are powered off.
- When replacing the current monitor with this monitor, be sure to change the PC settings for resolution and vertical scan frequency to those that are available for this monitor. Refer to the compatible resolution table (back of cover page) before connecting the PC.

Note

- When connecting multiple PCs to this product, refer to the User's Manual (on the CD-ROM).

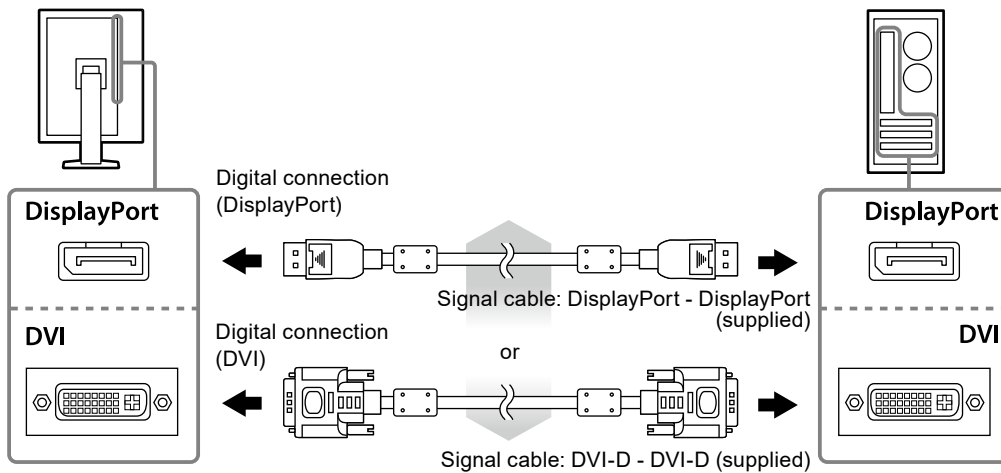
1. Turn the monitor screen 90° in clockwise direction.

The landscape position is the default monitor orientation. Turn the monitor screen ninety degrees to portrait position before installing it.

2. Connect the signal cables to the input signal connectors and PC.

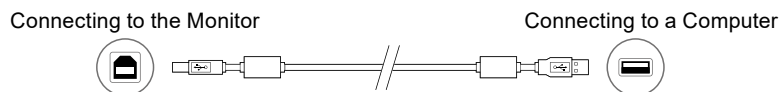
Check the shape of the connectors, and connect the cables.

After connecting the signal cable, tighten the screws of the connectors to secure the coupling.

3. Plug the power cord into a power outlet and the power connector on the monitor.

4. Connect the USB cable when using RadiCS LE or ScreenManager Pro for Medical.

5. Press to turn on the monitor.

The monitor's power indicator lights up green.

6. Turn on the PC.

The screen image appears.

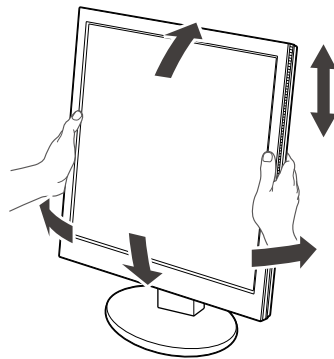
If an image does not appear, refer to "No picture" (page 6) for additional advice.

Attention

- Turn off the monitor and PC after using them.
- For the maximum power saving, it is recommended that the Power button be turned off. Turning off the main power switch or unplugging the power cord completely shuts off power supply to the monitor.

Adjusting the Screen Height and Angle

Hold left and right edge of the monitor with both hands, and adjust the screen height, tilt and swivel of the screen to the best condition for working.

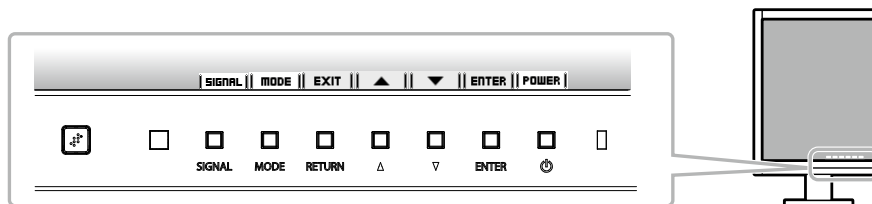
Attention

- Be sure that the cables are correctly connected.

Showing Button Guide

Press the front buttons (except ) button, the Button Guide appears above the button.

Note

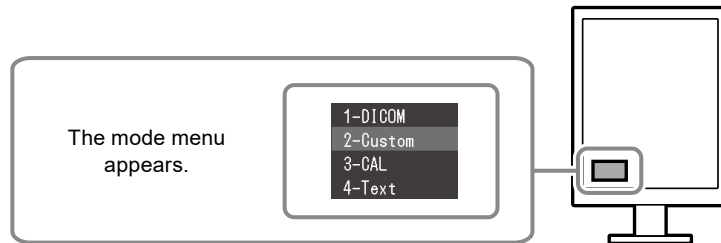
- The Button Guide will continue to appear while the Adjustment menu or mode menu is showing.
- The Button Guide is displayed differently depending on the selected menu or status.

Selecting Display Mode

This function allows easy selection of a desired mode according to monitor application.

√: Calibration is available.

Mode	Purpose	
1-DICOM	Available to display in DICOM mode.	√
2-Custom	Available for the setting according to your preference.	√
3-CAL	Displays the screen adjusted by calibration software.	√
4-Text	Suitable for displaying texts for word processing or spreadsheets.	-

Procedure

1. Press MODE.
The mode menu appears at the bottom left of the screen.
2. The mode among the list is highlighted in turn each time MODE is pressed.
You can switch the mode with ▲ or ▼ while the mode menu is displayed.

→1-DICOM←→2-Custom←→3-CAL←→4-Text←

Controlling Monitor Quality

The quality control software “RadiCS LE” (for Windows) is included with this monitor.

To calibrate the monitor and manage the calibration history, install RadiCS LE with reference to the RadiCS LE User’s Manual (on the CD-ROM). Use the RadiCS SelfQC function and built-in Integrated Front Sensor to perform periodic calibration and Grayscale Check for the monitor independently. For details, refer to the User’s Manual (on the CD-ROM).

Attention

- Connect the monitor to the PC with a USB cable when using RadiCS LE.

Note

- To perform advanced quality control that conforms to medical standards, use the optional monitor quality control software kit “RadiCS UX1”.

No picture

If a problem still remains after applying the suggested remedies, contact your local EIZO representative.

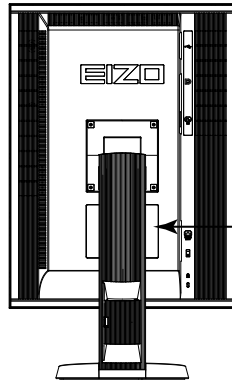
Problem	Possible cause and remedy
<p>1. No picture</p> <ul style="list-style-type: none"> Power indicator does not light. <hr/> <ul style="list-style-type: none"> Power indicator is lighting green. Power indicator is lighting orange. <hr/> <ul style="list-style-type: none"> Power indicator is flashing orange and green. 	<ul style="list-style-type: none"> Check whether the power cord is connected properly. Turn the main power switch on. Press . Turn off the main power, and then turn it on again a few minutes later. <hr/> <ul style="list-style-type: none"> Increase "Brightness" or "Gain" in the Adjustment menu. Switch the input signal with SIGNAL. Operate the mouse or keyboard. Check whether the PC is turned on. When Presence Sensor is set to "On", the monitor may have shifted to power saving mode. Come near to the monitor. <hr/> <ul style="list-style-type: none"> The device that is connected using DisplayPort have a problem. Solve the problem, turn off the monitor, and then turn it on again. Refer to the User's Manual of the output device for further details.
<p>2. The message below appears.</p> <hr/> <ul style="list-style-type: none"> This message appears when no signal is input. Example: <div data-bbox="316 947 627 1149" data-label="Image"> </div> <hr/> <ul style="list-style-type: none"> The message shows that the input signal is out of the specified frequency range. Example: <div data-bbox="316 1267 627 1532" data-label="Image"> </div> <p>fD: Dot Clock fH: Horizontal Scan Frequency fV: Vertical scan frequency</p>	<p>This message appears when the signal is not input correctly even when the monitor functions properly.</p> <hr/> <ul style="list-style-type: none"> The message shown left may appear, because some PCs do not output the signal soon after power-on. Check whether the PC is turned on. Check whether the signal cable is connected properly. Switch the input signal with SIGNAL. <hr/> <ul style="list-style-type: none"> Check whether the PC is configured to meet the resolution and vertical scan frequency requirements of the monitor (see "Compatible Resolutions"). Reboot the PC. Select an appropriate setting using the graphics board's utility. Refer to the manual of the graphics board for details.

No part of this manual may be reproduced, stored in a retrieval system, or transmitted, in any form or by any means, electronic, mechanical, or otherwise, without the prior written permission of EIZO Corporation. EIZO Corporation is under no obligation to hold any submitted material or information confidential unless prior arrangements are made pursuant to EIZO Corporation's receipt of said information. Although every effort has been made to ensure that this manual provides up-to-date information, please note that EIZO monitor specifications are subject to change without notice.

RadiForce® MX215

LCD-FarbMonitor

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WARNING
RISK OF ELECTRIC SHOCK. DO NOT OPEN.

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EIZO GmbH EC REP
EIZO Limited UK Responsible Person
EIZO AG CH REP