RadiForce® RX220

Color LCD Monitor

Instructions for Use: ■ Setup Manual

Important: Please read PRECAUTIONS, this Setup Manual and the User's Manual (on the CD-ROM) carefully to familiarize yourself with safe and effective usage. Please retain this manual for future reference.

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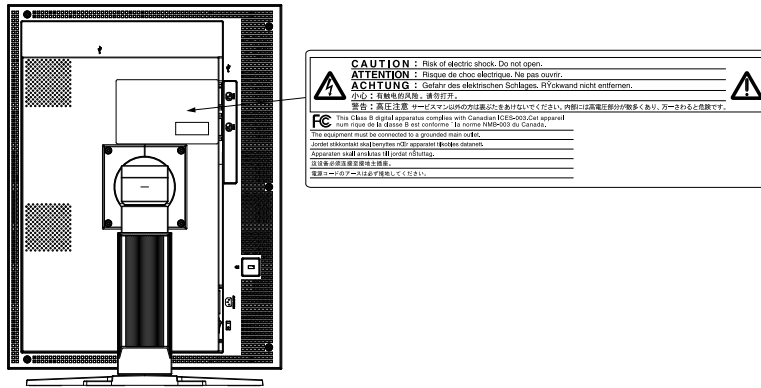
RadiForce® RX220

Color LCD Monitor

[Location of the Caution Statements]

Symbols on the unit

Symbol	This symbol indicates
	Main Power Switch Press to turn the monitor's main power off.
	Main Power Switch Press to turn the monitor's main power on.
	Power Switch Press to turn the monitor's power on or off.
	Alternating current
	Alerting electrical hazard
	Caution Refer to SAFETY SYMBOLS section in PRECAUTIONS.

No part of this manual may be reproduced, stored in a retrieval system, or transmitted, in any form or by any means, electronic, mechanical, or otherwise, without the prior written permission of EIZO Corporation.

EIZO Corporation is under no obligation to hold any submitted material or information confidential unless prior arrangements are made pursuant to EIZO Corporation's receipt of said information. Although every effort has been made to ensure that this manual provides up-to-date information, please note that EIZO monitor specifications are subject to change without notice.

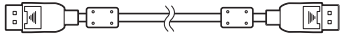
Package Contents

Check that all the following items are included in the packaging box. If any items are missing or damaged, contact your local dealer.


NOTE

Please keep the packaging box and materials for future movement or transport of the monitor.


- Monitor
- Power cord
- Digital signal cable : PP200

- Digital signal cable : FD-C39

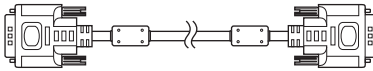
- EIZO USB cable : MD-C93

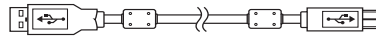- EIZO LCD Utility Disk (CD-ROM)
 - RadiCS LE (for Windows)
 - ScreenManager Pro for Medical (for Windows)
 - User's Manual
- Setup Manual (this manual)
- PRECAUTIONS
- Recycling Information

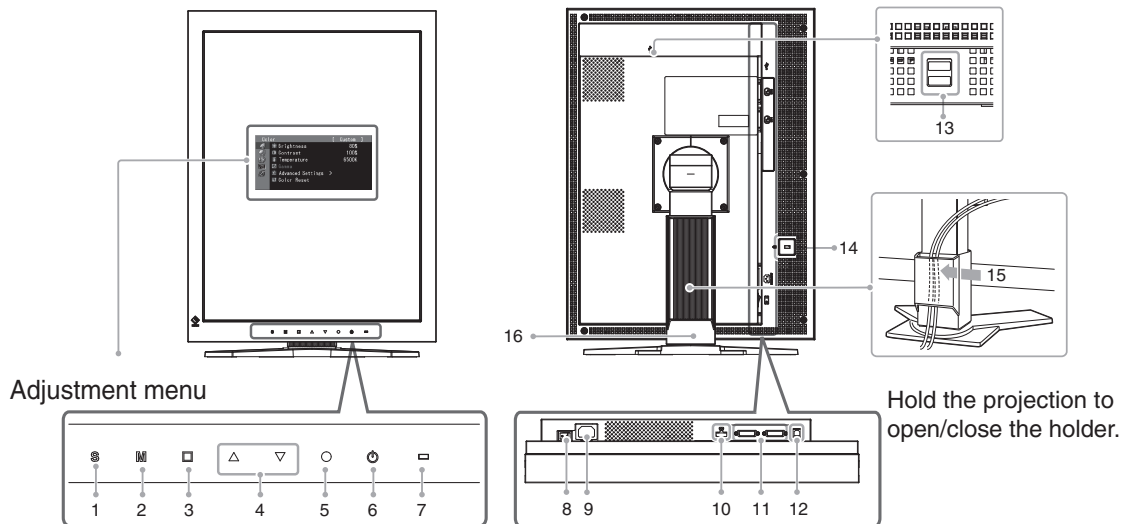Controls and Functions

1	S switch	Switches input signals for display when more than two PCs are connected to the monitor.
2	M switch	Allows you to switch the CAL Switch mode (page 5).
3	□ switch	Cancels the adjustment/setting or exit the adjustment menu.
4	Δ ▽ switch	<ul style="list-style-type: none"> • Chooses an adjustment item or increases/decreases adjusted values for advanced adjustments using the Adjustment menu. • Adjusts the brightness..
5	○ switch	Displays the Adjustment menu, determines an item on the menu screen, and saves values adjusted.
6	⏻ switch	Turns the power on or off.
7	Power indicator	Indicates monitor's operation status. Green: Operating Orange: Power saving Off: Main Power Switch / Power Switch off
8	Main Power switch	Turns the main power on or off.
9	Power connector	Connects the power cord.
10	Input signal connectors	DisplayPort Connector x 1
11	Input signal connectors	DVI-I Connector x 2
12	USB port (Up)	Connects the USB cable to use the software that needs USB connection, or to use USB Hub function.
13	USB port (Down)	Connects a peripheral USB device.
14	Security lock slot	Complies with Kensington's MicroSaver security system.
15	Cable holder	Covers the monitor cables.
16	Stand	Used to adjust the height and angle of the monitor screen.

Connecting Cables

Note

- Check that the monitor and the PC are powered off.
- When replacing the current monitor with a RX220 monitor, be sure to change the PC settings for resolution and vertical frequency to those which are available for the RX220 monitor referring to the resolution table (back of cover page) before connecting the PC.

Tips

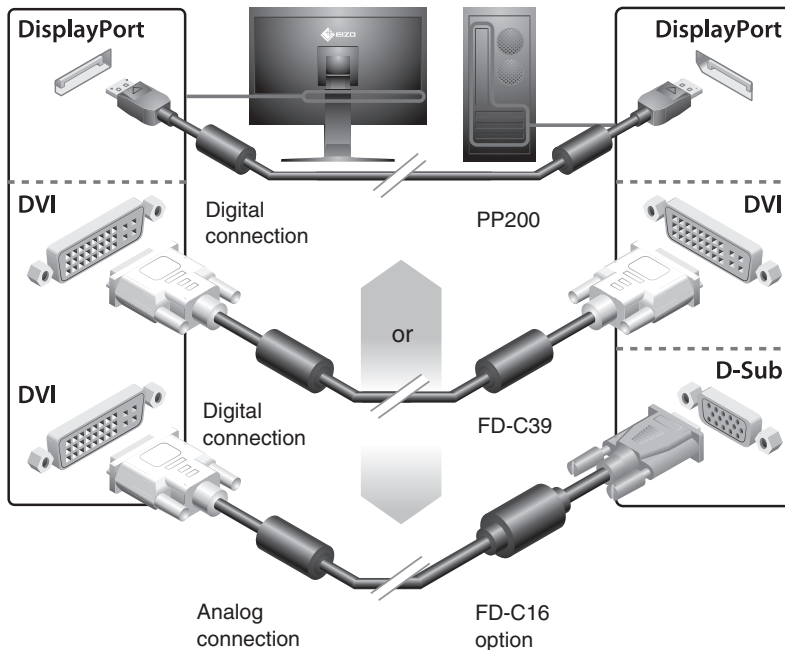
- When connecting more than two PCs to the monitor, refer to the User's Manual on the CD-ROM.

1 Turn the monitor screen 90° in clockwise direction.

The landscape position is the default monitor orientation. Turn the monitor screen ninety degrees clockwise to portrait position before installing it.

2 Connect the monitor to the PC with a signal cable that matches the connectors.

After connecting the cable connectors, tighten the screws of the connectors to secure the coupling.

3 Connect the USB cable when using RadiCS LE or ScreenManager Pro for Medical.

Connecting to the Monitor

Connecting to a Computer

4 Plug the power cord into a power outlet and the power connector on the monitor.

5 Press \odot to turn on the monitor.

The monitor's power indicator lights up green.

6 Turn on the PC.

The screen image appears.

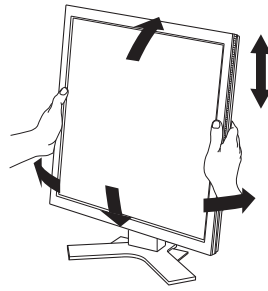
When turning on the monitor and PC for the first time with the analog signal, the Auto Adjustment function works to adjust the clock, phase, and display position automatically. If an image does not appear, refer to "No-Picture Problem" (page.6) for additional advice.

Note

- Turn off the monitor and PC after using them.
- For the maximum power saving, it is recommended that the Power button be turned off. Turning off the main power switch completely shuts off power supply to the monitor.

Adjusting the Screen Height and Angle

Hold left and right edge of the monitor with both hands, and adjust the screen height, tilt and swivel of the screen to the best condition for working.

Note

- Be sure that the cables are correctly connected.

Installing RadiCS LE

The quality control software "RadiCS LE" (for Windows) is included with this monitor. To calibrate the monitor and manage the calibration history, install RadiCS LE with reference to the RadiCS LE User's Manual on the CD-ROM.

Note

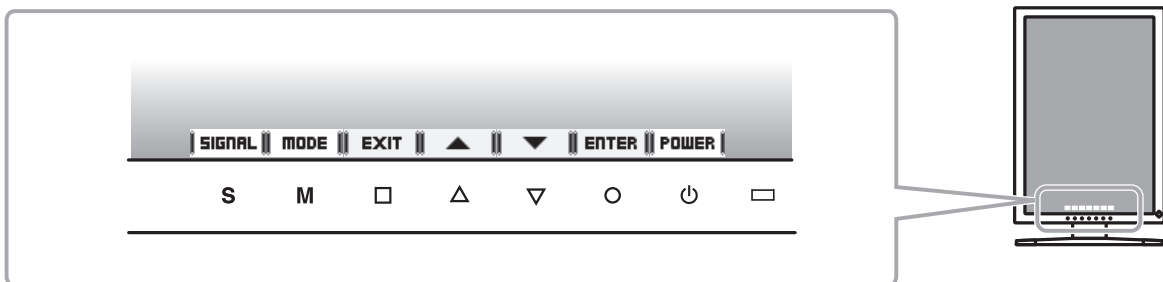
- Connect the monitor to the PC with a USB cable when using RadiCS LE.

Tips

- To perform advanced quality control that conforms to medical standards, use the optional monitor quality control software kit "RadiCS UX1".

Showing Button Guide

Press the front switches (except ⏻ switch), the button guide appears above the switch. (When using the monitor with "Landscape" position, the button guide appears next to the switch.)

Tips


- The button guide will continue to appear while the Adjustment menu or Mode menu is showing.
- The button guide is displayed differently depending on the selected menu or status.

Selecting Display Mode

CAL Switch allows you to select easily the adequate mode suitable for the monitor's application.

CAL Switch Mode

1-DICOM	Available for setting tailored to X-ray film color (clear base).
2-Custom	Available for the color settings according to your preference.
3-CAL	Displays the screen adjusted by calibration software.
4-Text	Suitable for displaying texts for word processing or spreadsheets.

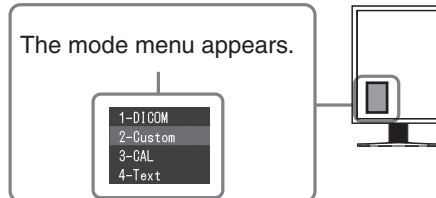
1 Press **M**.

Mode menu appears at the lower left of the screen.

2 The mode among the list is highlighted in turn each time **M** is pressed.

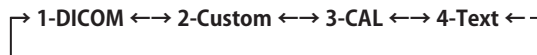You can switch the mode with Δ or ∇ while the mode menu is displayed.

The basic settings and adjustments are completed. For advanced settings/adjustments, refer to the User's Manual on the CD-ROM.

No-Picture Problem

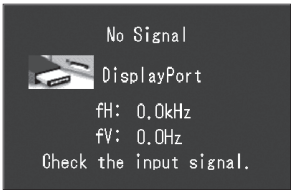
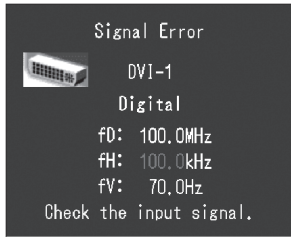
If no picture is displayed on the monitor even after the following remedial action is taken, contact your local dealer.

1. Check the Power indicator.

Symptom	Status	Possible cause and remedy
No picture	Power indicator does not light up.	<ul style="list-style-type: none"> • Check whether the power cord is connected correctly. • Turn off the main power, and then turn it on again a few minutes later. • Turn the main power switch on. • Press ϕ .
	Power indicator lights green.	<ul style="list-style-type: none"> • Set each adjusting value in [Brightness], [Contrast] and [Gain] to higher level.
	Power indicator lights orange.	<ul style="list-style-type: none"> • Switch the input signal with S . • Press a key on the keyboard or click the mouse. • Turn on the PC.

2. Check the error message that remains on the screen.



These messages appear when the input signal is incorrect even if the monitor is functioning.

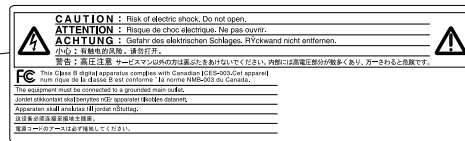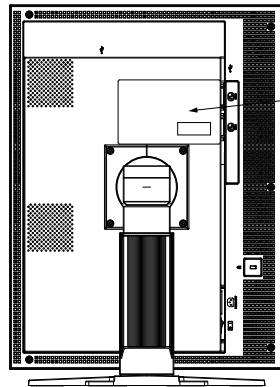Symptom	Status	Possible cause and remedy
 <p>No Signal  DisplayPort fH: 0.0kHz fV: 0.0Hz Check the input signal.</p>	The input signal is not received.	<ul style="list-style-type: none"> • Check whether the PC is turned on. • Check whether the signal cable is properly connected to the PC. • Switch the input signal with S .
 <p>Signal Error  DVI-1 Digital fD: 100.0MHz fH: 100.0kHz fV: 70.0Hz Check the input signal.</p>	The signal frequency is outside the specification. Incorrect signal frequency is shown in red.	<ul style="list-style-type: none"> • Reboot the PC. • Change the mode to an appropriate mode using the graphics board's utility software. Refer to the manual of the graphics board for details. fD: Dot Clock (Displayed only when the digital signal inputs) fH: Horizontal Frequency fV: Vertical Frequency

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